**Gebrauchsanweisung
Assembly Instructions
Instructions de montage
Istruzioni per il montaggio
Instrucciones de montaje
Montage-instructies**

Bettseitengitter

„Pivot-Rail“

Art.-Nr.: PRAIL8001

1.0 Vorwort:

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Assembly instructions
Bed side rails "Pivot-Rail"
Item number: PRAIL8001

1.0 Preface:

Dear customer,

we are pleased that you have chosen a quality product from *rehastage GmbH*. A good relationship with our customers is of great importance to *rehastage*. Customer proximity means: fast service, as little bureaucracy as possible and close cooperation with our customers. Seek advice from your treating doctor or therapist to ensure that this product is suitable for you and meets your needs.

⚠ **For your personal safety, please read these instructions carefully before using the product. If you have any problems understanding the information, warnings and instructions given here, please contact your dealer to avoid possible injury or material damage due to incorrect operation.**

Do not throw away these instructions for use even after the aid has been installed, so that you can use them later if necessary.

If you need spare parts or accessories or simply have a question about the product, we are here for you.

We want you to be satisfied with our products and our service. We at *rehastage* are therefore constantly working on the further development of our products. For this reason, changes in form, technology and equipment may occur in our product range. Therefore, no claims can be derived from the information and illustrations in these instructions for use.

2.0 Intended Use:

The "Pivot Rail" bed side rail is an aid that primarily serves as a fall guard and protection against falling out of bed. Due to the individually adjustable width of the side rail (66, 86 and 106 cm) as well as the possibility of mounting on both sides of the bed (left or right side), it can be easily adapted to the actual need.

Due to the possibility of swivelling down the bed side rail for frameless beds (e.g., Box Spring beds) as well as for standard beds, depending on the distance between the upper edge of the slatted frame and the upper edge of the bed frame, it is not in the way during nursing activities.

At the same time, it is an aid that enables the user / patient to get into a sitting position without outside help or to act as a support when getting in and out of bed. It is used by nursing staff, caring relatives or by the user / patient him/herself. It enables the user / patient to.

• Indication:

The bed side rail "Pivot Rail" is intended for use by adult persons / patients who do not suffer from the contraindications listed below. It serves as fall protection and protection against falling out of bed for patients at risk.

It can be used as a raising aid for users and / or patients with generally reduced or impaired mobility.

The provision of raising aids is indicated when a loss of balance in position between raising the upper body and achieving a safe sitting posture cannot be maintained. For example, it must be possible to put sufficient weight on one arm and there must be sufficient trunk control and stretching ability of the legs to enable the upper body to be raised largely independently.

• Contraindications:

⚠ The bed side rail "Pivot Rail" should not be used by people with Alzheimer's disease or dementia who take sedatives, are confused or infirm. There is an increased risk of entrapment and injury here.

⚠ For people trying to climb over the product, there is an increased risk of injury from falling.

⚠ The device is NOT intended for use by children.

⚠ **It is essential to observe the further warnings and precautions listed under point 5!**

The bed side rail "Pivot Rail" is a class I medical device according to Regulation (EU) 2017/745 of the European Parliament on medical devices.

3.0 Scope of delivery:

Remove all parts from the packaging and check them for completeness (see Figure 1).

Part A	Bett side grille	1 piece
Part B	Base tubes with safety strap	2 pieces
Part C	Connecting tubes	2 pieces
Part D	Screws M6x40	4 piece
Part E	Screws M6x80	2 piece
Part F	Washer Ø6,5	6 piece
Part G	Nut M6	2 piece
Part H	Wrench	1 piece

Assembly instructions
Bed side rails "Pivot-Rail"
Item number: PRAIL8001

Part J	Allen key	1 piece
Part K	Textile cover (bag, pre-assembled)	1 piece

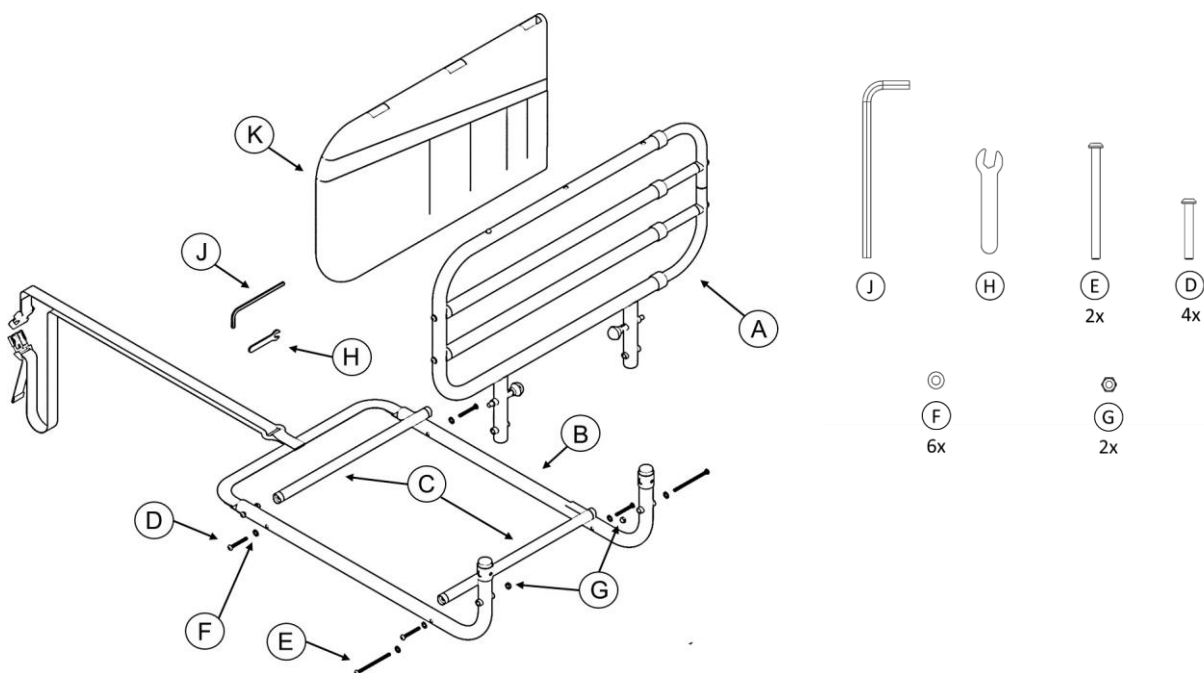
Figure 1

4.0 Assembly and installation instructions, operation:

4.1 Using the tool provided (J), attach C to D and F to B as shown in Figure 2.

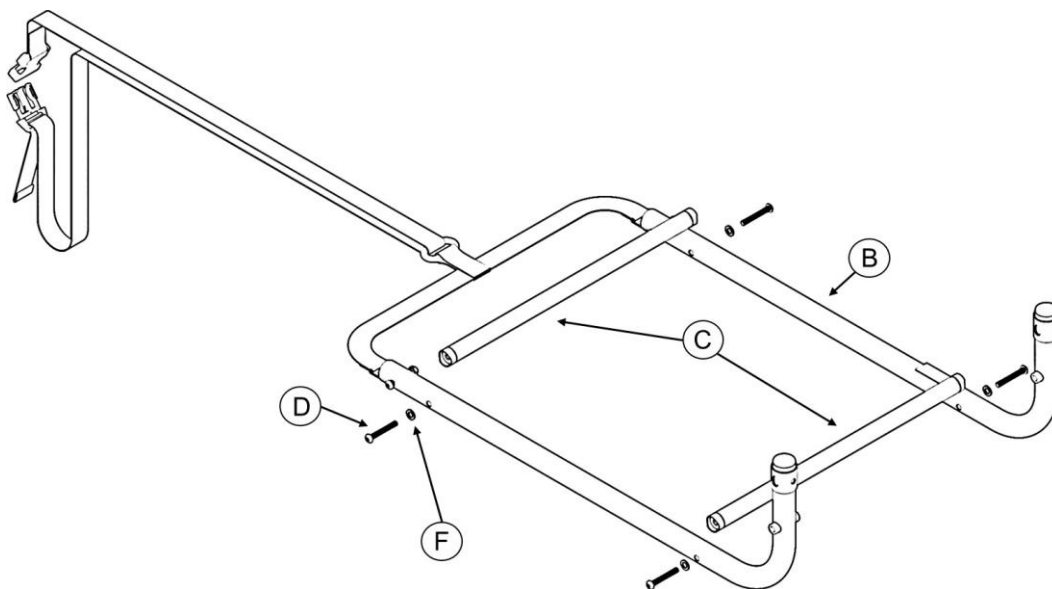
Figure 2

4.2 Determine on which side of the bed the product should be installed as shown in FIGURE 3. The adjustable length side of the bed side rail should always face the foot end of the bed.
Using the tools provided (H and J), attach A and B with the fasteners (E, F and G) as shown in FIGURE 3.

Assembly instructions
Bed side rails "Pivot-Rail"
Item number: PRAIL8001

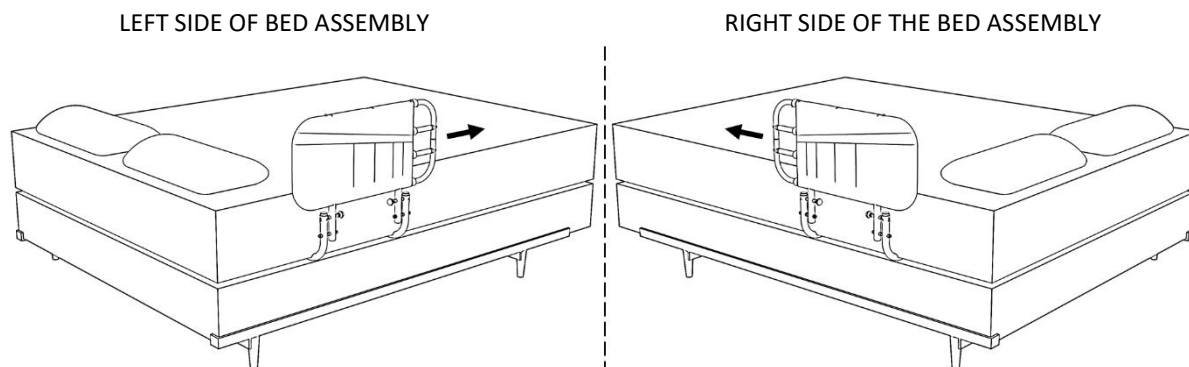
Figure 3

4.3 Using the provided tools (H and J), attach A and B with the fasteners (E, F and G) as shown in FIGURE 4.

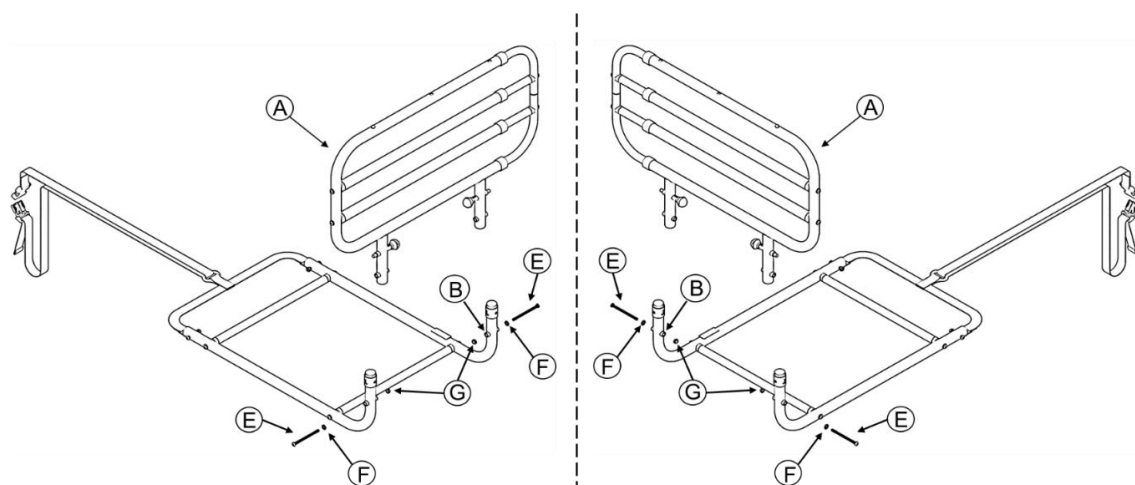
Figure 4

4.4 Open the lock of the SAFETY Harness and guide it around the bed frame / slatted frame. Put the lock back together and test whether the lock is firmly engaged. Then tighten the SAFETY Harness so that the bed side rail is tight against the side of the mattress as shown in Figure 5.

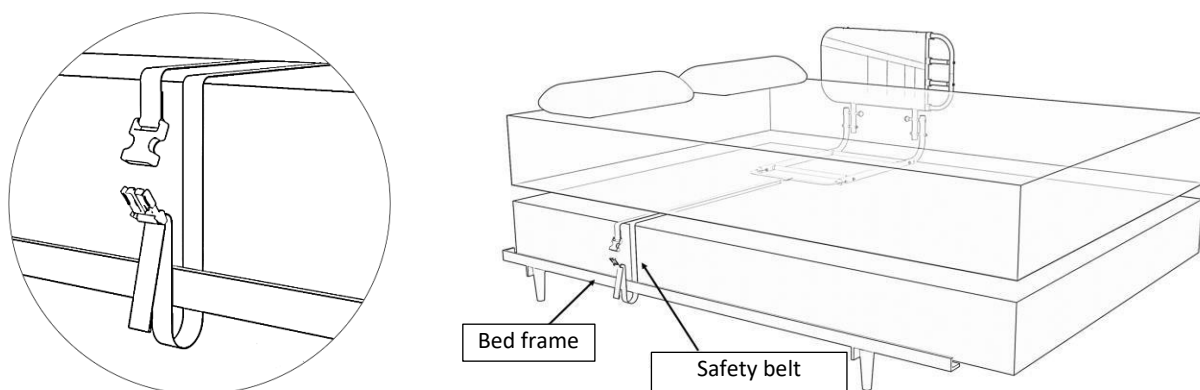
Figure 5

Assembly instructions
Bed side rails "Pivot-Rail"
Item number: PRAIL8001

4.5 Operation:

To extend the bed side rail, press the spring button and pull the extension side of the bed side rail outwards, as shown in figure 5 a.

To lower the bed side rail, pull the knobs and turn them 90 degrees, as shown in figure 5 b and 5 c.


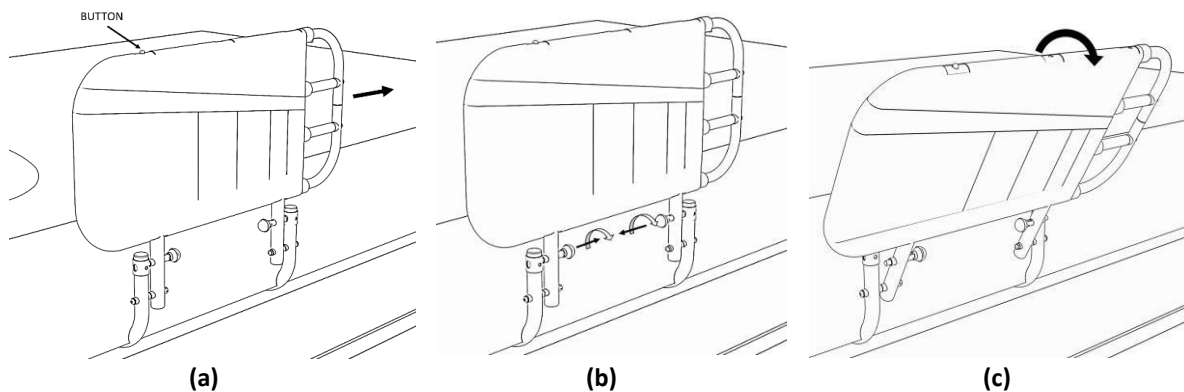
Figure 5

5.0 Precautions / Warnings:

⚠ For safety reasons, it is important that the following conditions are met during installation and use in order to avoid the risk of injury or entrapment and any resulting risk of suffocation.

5.1 Dimensions and distances to be observed (based on standard DIN EN 60601-2-52):

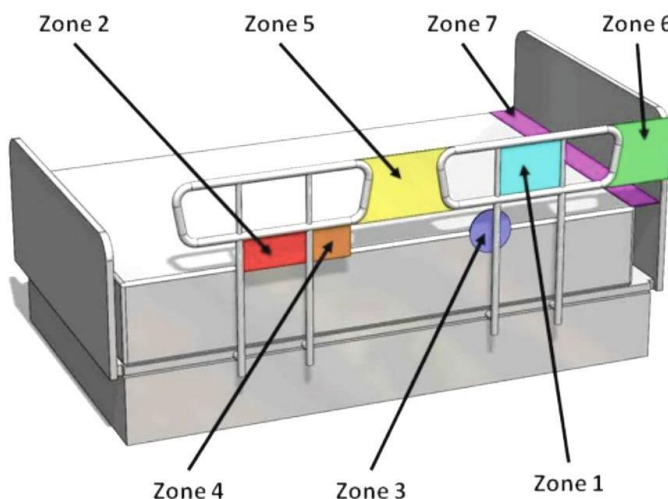Zone 1: Inside the Bed Side Rail - Any open space inside the bed side rail can be a risk for head entrapment. The maximum allowable space is 12 cm (4.75 in.).

Zone 2: Below Bed Side Rail - The clearance between the bottom edge of the bed side rail and the mattress can allow dangerous head entrapment. The maximum allowable clearance is 12 cm (4.75 in.).

Zone 3: Between the bed side rail and the mattress - This area between the inside of the bed side rail and the mattress, if too large, can be a risk for head entrapment. The maximum allowable distance is 12 cm (4.75 in.).

However, if possible, the bed handle should be tight against the mattress.

Zone 4: Below the bed side rail, at the ends - A gap between the mattress and the lowest cross member of the bed side rail poses a risk of entrapment. The recommended gap is 2.5 - 6 cm (0.65 - 2.375 in.).

Zone 5: Between Split Bed Side Rails - When multiple bed side rails are used on the same side of the bed, the space between bed side rails may pose a risk of head, neck, or chest entrapment. The distance between bed side rails should be min. 32 cm (12.5 in.)

Zone 6: Between the end of the bed side rail and the head of the bed - The recommended distance between the end of the bed side rail and the head of the bed is at least 32 cm (12.5 in.), otherwise there is a risk of trapping the head, neck or chest.

Assembly instructions
Bed side rails "Pivot-Rail"
Item number: PRAIL8001

Zone 7: Between the head or foot of the bed and the end of the mattress - The size of the mattress should fit the bed frame or base exactly. If the distance between the head or foot of the bed and the end of the mattress is too great, the risk of entrapment increases.

5.2 Mattress properties and requirements:

There is an increased risk of entrapment and resulting injury when using mattresses that are too soft or too thin.

To ensure that the existing mattress is suitable for use with this product, test the firmness of your mattress by pressing a fist-sized object weighing approximately 5.4 kg (12 lbs) onto the edge of the mattress. If the sinking depth is more than 5 cm (2 inches), the mattress used is too soft and the bed rail should not be used.

5.3 Warnings, Notices and Exclusions:

⚠ The instructions accompanying the product must be followed carefully to ensure that the product is properly installed and assembled. Failure to follow the instructions may result in serious injury to the user(s) of the product.

⚠ The product is not intended to support the full weight of a person. This product is only intended as a balance aid and support when sitting down and getting up from bed.

⚠ The bed side rail "Pivot Rail" must not be used on children's beds, bunk beds, water beds or inflatable beds.

⚠ Never attach the bed side rail "Pivot Rail" on the handle side additionally to the bed, slatted frame or bed frame, this can lead to serious injuries to the user(s) due to jamming.

⚠ Check the positioning of the bed side rail "Pivot Rail" regularly to identify areas where entrapment is possible at an early stage.

⚠ With electrically adjustable bed sections, make sure that you do not trap your limbs when raising or lowering the bed.

⚠ Make sure that children do not play with or hang onto the product.

⚠ Additional safety measures should be taken for persons belonging to certain risk groups. These include persons with a mental impairment (including persons who are under the influence of medication), very tall persons or persons who are very overweight (which could alter the gap between the bed side rail "Pivot Rail" and the mattress), as well as persons who are generally very restless when awake or asleep.

⚠ If you have an illness or injury, or are taking medication that affects your sense of balance or your ability to stand or sit, please consult your physician first before using this product. Improper use of the product may result in serious injury or damage.

• **DISCLAIMER:** Neither Stander Inc. nor the manufacturer/distributor shall be liable for any damage or injury that may result or have resulted from the improper use or installation of the product.

• In the event of assembly problems, please contact the place of sale (specialist trade) or directly to:

Rehastage GmbH
Artlandstraße 80
49610 Quakenbrück / Germany
gm@rehastage.de

• In the event of serious adverse events (deterioration of the user's condition) caused by the use of our product, we are obliged to report this to the Federal Institute for Drugs and Medical Devices. Therefore, we ask you to inform us directly in such a case:

Rehastage GmbH
Artlandstraße 80
49610 Quakenbrück / Germany
gm@rehastage.de

6.0 Cleaning instructions

Clean the "Pivot Rail" bed side rail regularly with a damp sponge, mild soap and a household disinfectant. Dry it with a dry soft cloth. Do not use abrasive cleaners such as scouring cream or powder, this may destroy or damage the surfaces and threads.

7.0 Reuse / service life

Due to its simple and solid construction, the bed side grille "Pivot Rail" is suitable for reuse. The number of reuses depends on the state of use of the device. Before each reuse, the "Pivot Rail" bed side grille must be hygienically prepared and the technical and functional safety of the product checked and, if necessary, restored.

Assembly instructions
Bed side rails "Pivot-Rail"
Item number: PRAIL8001

The assembly has to be carried out as described under point 4.

If the product is used as intended in accordance with the specifications, the service life is expected to be 5 years.

However, the effective service life may vary depending on the frequency and intensity of use as well as on the care and general condition.

8.0 Technical data:

Width of the bed rail:	Adjustable in widths 66, 86 and 106 cm
Frame height:	58.4 cm
Own weight:	6,8 kg
Max. User weight:	300 lbs (136 kg)
Max. Load in use:	Depending on the mounting conditions

9.0 Disposal / recycling of materials

If the aid was provided to you free of charge, it may not belong to you. If it is no longer needed, follow the return instructions provided by the organization that provided you with the aid.

The following section describes the materials that were used for the resource, including information on the disposal or recycling of the aid and its packaging. Special regulations may also apply locally with regard to disposal or recycling; these must be taken into account during disposal.

- steel: Bed side rail "Pivot Rail", assembly material and tools
- plastic: belt, packing material
- cardboard box: Overwrapping

Disposal should be done through a disposal company or public disposal site.

10.0 Warranty

In addition to the statutory warranty, rehastage offers a warranty on this product for manufacturing or material defects of 2 years from the date of purchase. The warranty applies only to the first user of the item. Please keep the invoice, which is your proof of purchase. The warranty does not cover defects due to wear and tear, improper handling or commercial use of the item. The warranty is void if the item is repaired or constructively altered by anyone other than rehastage itself or a dealer authorized by rehastage. The warranty is void if repairs or modifications or replacement of individual parts that are not original rehastage spare parts are made by anyone other than the seller or supplying company. During the warranty period, rehastage shall decide whether to repair the device free of charge or replace it free of charge.

11.0 Explanation of symbols used, markings and information to be provided

				
Manufacturer, name and address of manufacturer	Manufacturer's authorised representative within the European Community	Medical device	Attention! Information such as warnings or precautions	Follow the instructions for use!

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