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EN 1 Whilst seated, measure the circumference of your bottom (in inches or cm), as indicated in FIG.1.
2 On the inflation table FIG.2, cross-reference the circumference of your bottom (in inches or cm) with your weight.
Example (FIG.2): Weight: 209lbs/95 kg; Bottom circumference: 50 inches/125 cm
The inflation value is 3 (which matches indicator light 3 on the pressure gauge).
NB: If your inflation level is on the limit between two zones, opt for the higher of the two.
POLYAIR® cushion is available in several sizes (recommendation board) FIG.3.

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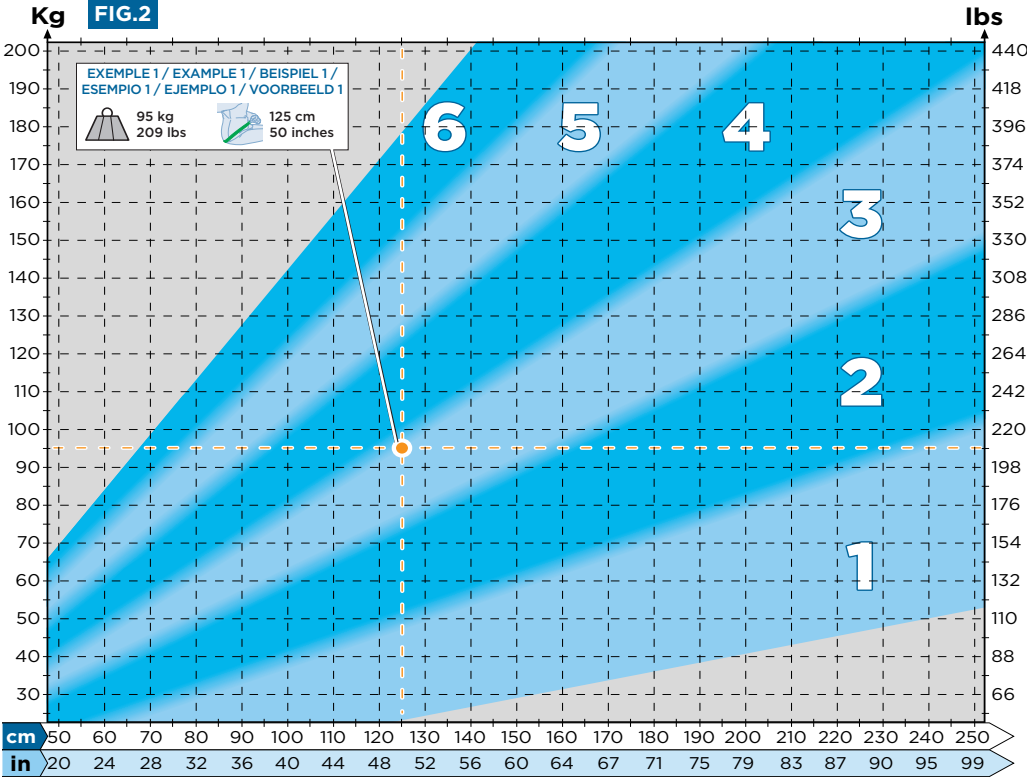
EN WHY INFLATING WITH A PRESSURE GAUGE MANOMETER?
Maladjusted inflating impairs the air cells cushions efficiency. An over-inflated or under-inflated cushion can lead to pressure sore outbreak. Since the inflating variations inside of each compartment (few mm/hg) can not be detected through hand feeling, the sole way to inflate properly air cells cushions is to use inflating tools such as pressure gauge manometer. Only the manometer is able to take into account the weight and contact surface of patient with the cushion. This method showing constant results grants an efficient prevention.

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EN Air cell system inflated with a pressure gauge.
Well-distributed support reducing transcutaneous pressure in the risk zones. Effective prevention.

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(EN) **INFLATING SYSTEM PRESSURE GAUGE - MANOMETER**
1 On/Off button
2 Inflation level indicator lights
3 Deflation button
4 Pre-inflation indicator
5 Inflation bulb
6 POLYAIR® connection valves

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(EN) The inflating instructions below are detailed for 2 chambers air cells cushions. In case of single chamber air cells cushions, the inflation method is identical for the single chamber.

1 - PRE-INFLATION
Why should you use pre-inflation?
Pre-inflation is an essential step for adjusting the cushion.
The pre-inflation phase enables the pressure levels of both of the cushion's compartments to be balanced better, in order to facilitate adjustment subsequently.
Pre-inflation procedure FIG.5.
Pre-inflation must be carried out WITHOUT having the patient sitting on the cushion.
Start up the pressure gauge by pressing On/Off button **1** FIG.4. Pre-inflation indicator **2** FIG.4 will flash on and off. Connect the pressure gauge using one of the 2 valves and pre-inflate each compartment, one after the other, until the pre-inflation indicator lights up (pre-inflation indicator **1** FIG.4). Should this value be exceeded, deflate by pressing deflation button **3** FIG.4.

2 - POSITIONING OF THE PATIENT (during the inflating stages)
In order to achieve the correct inflating, please follow the instructions for the positioning below :
A The wheelchair has a straight-backed position.
B The arms should not rest on the armrests FIG.6
C The feet are positioned on the footrests, which have been set in order that the legs take a right angle position FIG.6

3 - ADJUSTING THE PRESSURE FIG. 7
A Position the patient on the preinflated cushion.
B Connect the pressure gauge to one of the cushion's 2 valves, making sure that the connection is properly placed on the valve. Switch the pressure gauge back on if it is switched off.
C Check the pressure of the compartment and adjust it so that you obtain the inflation value determined previously.
For this, there are two 2 possible cases:
C-1 The pressure gauge reading is higher than the inflation value targeted or all of the indicator lights will flash on and off (indicating a pressure level that is too high to be measured); in this case, deflate the cushion gradually using deflation button **3** FIG.4, not forgetting to get the patient to get up several times during this process in order to enable the air to circulate in the compartment.
C-2 The pressure gauge reading is lower than the determined inflation value; in this case, gradually inflate using inflation bulb **5** FIG.4, not forgetting to get the patient to get up several times during this process in order to enable the air to circulate in the compartment.
Follow the same procedure for the second compartment. To switch off the pressure gauge, press On/Off button **1** FIG.4 and hold it down.

CHECKING AND REINFLATING THE CUSHION
NB. Each time you check or reinflate the cushion, make sure you comply with the positioning described in section 2. As is the case with most inflatable items, it is recommended that you check the pressure periodically (every 1 to 2 weeks on average).

GARANTEE
Two years against any defaults in manufacture, as of the date of purchase by the customer. In the event of default, please contact your retailer, and give them the product serial number. The warranty does not cover misuse of the support.

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© SYSTAM® - Premier marque CC : 2010. Les marques SYSTAM® et Polair® sont des marques déposées (Logos France et International) Document non contractuel. La so- ciété SYSTAM (System Assistance Medical) se réserve un droit de modification sans préavis.