
COJIN ANTIESCARAS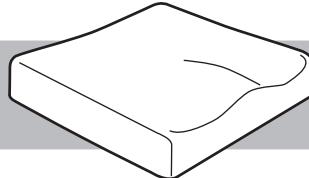

VISCOFORM

C 7A: 42 x 42 x 8 cm

C 7B: 45 x 42 x 8 cm

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Cushion made of Polyurethane viscoelastic memory foam, anatomic shape moulded with polyurethane breathable fabric cover that is waterproof.

PREWARNINGS AND PRECAUTIONS

For reasons of hygiene, all medical devices used for the prevention of bedsores should be reserved for the exclusive use of one patient.

In the event of incontinence, the cover and eventually the cushion should be cleaned. It is advisable to purchase a second protective cover for use while the first is being washed, so as to avoid direct contact with the surface of the cushion.

Do not place any other materials or objects between the cushion and the patient's clothing, apart from the protective cover. Seating cushions should not be used to support other parts of the patient's body (only the buttock zones).

The ideal temperature range for use is between -5° C and 45° C.

STORAGE

- For good longevity, it must be stored in a warm dry place, away from heat and light.
- Do not expose to direct heat.
- Store flat.

CONDITIONS OF USE

Because of the cushion's anatomical shape, for a good seating position in a chair, ensure that:

- The size of the cushion suits the chair. It is preferable not to use the cushion on a steeply sloping surface.

- Ensure that the positioning of the front, back, top and bottom are correct.

MAINTENANCE / CLEANING

Do not iron. Do not use corrosive products or abrasive materials. **Cushion:**

Wash in soapy water, rinse in clear water. Dry in open air. **Cover:**

Wash at 40°. Do not use bleach, do not iron. Dry clean with P-type solvent. Tumble dry at moderate temperature. Compatible with hospital requirements (washable up to 90° C)

DISINFECTING / DECONTAMINATING

Use cold spray products (ammonium quaternai-re-glutaraldheydes, etc.)

ENVIRONMENT

Do not burn or dump in the countryside. 0% CFC.

SAFETY

ViscoForm cushions comply with the FIRE safety regulations in force.

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